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**ENGLISH**

### Symbols and Their Explanations

-  Read the user manual before use!
-  Warning!
-  It is not sterile!
-  Do not reuse !
-  Store in a dry place!
-  Protect from sunlight!
-  Humidity limitation symbol: Store in environments not exceeding 0-80% humidity!
-  Manufacturer
-  CE marking according to (EU) 2017/745 Medical Device Directive

### References

**(EU) 2017/745** Medical Device Regulation

**EN ISO 20417:2021** Medical devices - Information to be provided by the manufacturer

**EN ISO 15223-1** Medical devices - Symbols to be used in the information to be submitted by the manufacturer - Part 1: General requirements

**EN ISO 17664-1** Processing of health care products - Information provided by medical device manufacturers for the processing of medical devices - Part 1: Critical and semi-critical medical devices

**EN ISO 17665-1** Sterilization of healthcare products - Moist heat - Part 1 - Requirements for the development, validation and routine control of a sterilization process for medical devices

**EN ISO 17665-2** Sterilization of health care products - Moist heat – Part 2 - Requirements for the development, validation and routine control of a sterilization process for medical devices

**EN ISO 14630:2012** Non-active surgical implants - General specifications

**ISO 11607-1:2019** (Packaging for Terminally Sterilized Medical Devices - Part 1: Requirements for Materials, Sterile Barrier Systems and Packaging Systems)

### Product Description

Spinal System implants and related components should only be used for bone fixation by surgeons with the necessary practice competencies and full experience and certification in the medical field. Spinal System implants are made of ISO 5832-1 certified stainless steel, ISO 5832-3 certified titanium alloy and are available in various sizes and designs. Titanium and stainless steel are completely biocompatible, corrosion resistant and non-toxic.

Ortimplant Spinal System implants are sold non-sterile and all devices are disposable.

These products, components, treatment kits and tools are manufactured for the purpose of medical interventions for the treatment of patients and may not be used for any other purposes. Otherwise, Ortimplant cannot be held responsible.

**Note:** The size, type and dimensions of the implant and accessories to be used should be determined by the surgeon who will perform the application according to the bone area where the "application sets" suitable for the selected implant and accessories will be used, the condition of the disease, the gender of the patient and the age of the patient.

Before using Ortimplant products, surgeons and their assistant team should read these statements carefully. All necessary documents and materials must be provided by the authorized distributor. Accordingly, the surgeon should know the risks that may arise as a result of the use of these products and inform the patient about these risks as much as possible. Product implantation should be performed by professionals trained and experienced in medical spinal system practice.

Spinal System implants produced by Ortimplant are produced in different shapes and sizes according to human bone structures, bone anatomical shapes, fracture configurations and intended use. Each product is designed according to the specific bone problem.

#### **Types of Spinal System Implants and the sets they are used in;**

- ✓ Spinal system implants may include:
- ✓ Spinal Pedicle Screw System Instrument Set
- ✓ Anterior Cervical Plate System Instrument Set,
- ✓ Anterior Thoracic & Lumbar Plate System Instrument Set.

Specialists are responsible for the consequences of using the wrong implant, not meeting the safety criteria in this user guide, or using inappropriate surgical techniques. In such cases, Ortimplant and its authorized distributor are not responsible to the patient and surgeon.

Ortimplant Spinal System implants can only be used with Ortimplant application sets and kits. The surgeon performing the application should definitely avoid using implants and application sets of other companies with Ortimplant products. Before performing such an application, the performing surgeon must make sure that the plate, screws, hand tools and tools will be sufficient to ensure bone alignment. Otherwise, the surgeon is responsible for the surgery and postoperative complications. Ortimplant and its distributors are not responsible to patients and surgeons under these circumstances.

During the preparation phase before the surgery, the surgeon must ensure that the plate implants, application sets and implants he will use are compatible with each other. ORTİMPLANT produces Spinal System implants for normal human anatomy and bone structures; During surgery, implants should not be bent more than twelve degrees by the surgeon to fit the damaged bone anatomical structure. Repeated bending and corrections made by the surgeon during surgery may cause implant failure due to early development of metal fatigue. Reuse may lead to inadequate treatment of the patient. In such cases, the surgeon is responsible. ORTİMPLANT and the distributor cannot be held responsible. Ortimplant and its distributors are not responsible for the negative consequences of surgical application in case of any complications.

**Note:** Spinal System implants are manufactured from ISO 5832-1 stainless steel and ISO 5832-3 titanium alloy certified raw materials. Do not use titanium alloy and stainless steel implants in the same place.

**Note:** To get the best result; Do not use Ortimplant Spinal System implants together with parts produced by other companies. Do not reuse Spinal System implants due to metal fatigue and risk of infection.

**Note to Surgeon:** Even if the physician is a knowledgeable intermediary between the patient and the company, the important medical information provided in this document must be disclosed to the patient.

#### **Intended Use**

All Spinal System Implants; Depending on the type, model, size and surgeon's choice, they are used in the treatment of spinal bone fractures in the body, scoliosis, kyphosis, spondylolisthesis, spondyloarthrosis and congenital or acquired bone-joint disorders.

#### **Indication Information**

It is used in the treatment of vertebral fractures, scoliosis, kyphosis, spondylolisthesis, spondyloarthrosis and other congenital or acquired spinal system disorders. Ortimplant spinal system implants are produced according to different anatomies and different sizes of spinal bone structures of the human body.

**Indications include:**

- ✓ Fixed angle, multi-angle screws, Hooks and Accessories (5 / 6mm rods, locking set screws, connectors, cross-connection plates) are used for fixation of cervical / thoracic / lumbar vertebrae with a posterior approach. It should be used with the Ortimplant Posterior Spinal System instrument set.
- ✓ Anterior cervical plate 1-2 and Accessories (4 / 4.5mm locking screws, locking apparatus, 3.5mm cortical screws) are used to fix the cervical vertebrae with the anterior / anterolateral approach. It should be used together with Ortimplant Anterior Cervical Plate Spinal System Set.
- ✓ Anterior Thoracic-Lumbar Plate and Accessories (6.5mm bolt, 6.5mm screw) are used for thoracic/lumbar vertebra fixation. It is used together with the Ortimplant Anterior Thoracic and Lumbar Plate Spinal System Instrument Set.

**Note:** The size, type and dimensions of the implants and accessories to be used in surgery and the "application sets" suitable for the selected implants and accessories should be determined by the surgeon who will perform the application according to the bone area to be used, the structure of the bone, the condition of the disease, the gender of the patient and the age of the patient. The Ortimplant catalog contains information about the Spinal System to be used with implants.

Planning the surgical procedure should be based on detailed evaluation of the patient's x-rays. In this way, the appropriate Spinal System implant selection is determined by the surgeon by calculating the size and finish combinations. Pre-operative planning templates are available from the manufacturer. The patient's allergic sensitivity to the implant material should be determined. Spinal System of different sizes should be available at the time of surgery. Before placing the appropriate implant, the surgeon must ensure that the implant size and type are compared to the label to use the correct size implant. Excess labels inside the box; for reporting and ordering purposes.

**Note:** Spinal System implants, components and application sets suitable for the implant should be selected by the surgeon and his team before the operation, and their operation should be checked and verified.

The surgeon and his team are responsible for selecting the appropriate implant and its components for the patient and the application sets suitable for these implants and components. Ortimplant and distributors cannot be held responsible for any problems arising from the selection and application of inappropriate implants, components and application sets.

**Contraindication Information**

The following situations include, but are not limited to, contraindications;

- ✓ Development of active infection or risk of infection (such as immunodeficiency),
- ✓ Indicators of local inflammation,
- ✓ Increased fever and white blood cell mass,
- ✓ Circulatory disorders and occlusions that prevent adequate blood circulation in the fracture or surgical area,
- ✓ Morbid obesity,
- ✓ Pregnancy (until the patient's suitability for surgery conditions is confirmed by experts),
- ✓ Mental illnesses,
- ✓ Some mental or neuromuscular diseases that may cause complications or fixation failure in the postoperative period,
- ✓ Patients with congenital serious anatomical disorders,
- ✓ The presence of another disease, the use of a drug or device related to it, can potentially reduce or prevent the benefit of the function of implants (e.g., progressive streptococcal infections, congenital functional abnormality, unexplained elevation of semimetal value, other disabling conditions).
- ✓ Rapidly progressive joint disease, bone erosion, osteopenia and/or osteoporosis. Osteoporosis and osteopenia are contraindications that may limit the degree of healing due to poor stabilization and/or mechanical fixation.

- ✓ Cases of titanium and stainless steel metal allergy,
- ✓ Situations where bone graft or fusion is unnecessary,
- ✓ Situations where the implant components selected for use are too large or too small for the bone area to be applied,
- ✓ Patients who do not have sufficient soft tissue, adequate bone stock, or sufficient bone quality to cover the operation area,
- ✓ Patients expected to have low physiological performance,
- ✓ Patients who are not willing to follow postoperative instructions,
- ✓ Using products in situations not specified in the indications,

**Note:** Due to the high risk of unexpected metal corrosion inside the body, two different metal components should not be used together.

#### **Potential adverse events and complications**

All known scientific safety warnings regarding bone surgery should be taken into account by the surgeon and the team during the implantation of the implants and its components. Potential safety warnings with implantation of the implant and its components are listed below, but are not limited to;

- ✓ Early or late loosening of spinal system implants and components,
- ✓ Possibility of disassembly, bending and/or breakage of all components within the body,
- ✓ Development of foreign body reaction (allergy) to implants, rash formation, corrosion of implants (cracks, friction and/or general corrosion)
- ✓ In patients without adequate soft tissue cover, skin penetration due to pressure on the operation side, increase in fibrous tissue, development of skin necrosis and/or pain, and inflammation of surrounding tissues and bone may occur. Improper use of instruments or incorrect placement of the implant may cause damage to surrounding tissue and nerves.
- ✓ There may be inflammation of surrounding tissues and bone. Improper use of instruments or incorrect placement of the implant may cause damage to surrounding tissues and nerves.
- ✓ After the surgery, changes in bone smoothness, limb weakening and/or length loss in the relevant body part should be monitored by the surgeon and the patient,
- ✓ Development of local and/or general infection,
- ✓ Paralysis (complete or partial), dysesthesia, hypersensitivity, numbness, paresthesia, appearance of radiculopathy and/or pain, depersonalization, loss of neurological function such as neuroma development, loss of consciousness, tingling sensation and/or visible defects,
- ✓ Disorders of bladder fullness, urinary retention and other urinary system disorders,
- ✓ Scar formation can cause nervous system problems or pressure and pain around the nerve,
- ✓ Fracture, microfracture, resorption, development of bone defect or penetration of bone,
- ✓ Pseudoarthrosis, late and poor union of the fractured bone where the surgical operation was performed,
- ✓ Increased/decreased mobility and function in the adjacent joints of the operated bone,
- ✓ The patient cannot continue his daily activities,
- ✓ Loss or decrease in bone density (probably a result of a decrease in the load on the bone),
- ✓ Complications of the grafted area, including pain, fracture or wound healing problems,
- ✓ Intestinal obstruction or loss of control, gastritis or other gastrointestinal tract disorders
- ✓ Deep vein thrombosis,
- ✓ Bleeding, hematoma, seroma, edema, hypertension, vascular occlusion, stroke, excessive bleeding, phlebitis, tissue decay, wound dehiscence, vascular injuries or other cardiovascular system disorders,
- ✓ Incorrect placement of Spinal System Implants in intervertebral disc areas,
- ✓ Reproductive system disorders, sexual dysfunction, progression of respiratory problems (e.g. pulmonary embolism, pneumonia, bronchitis),

- ✓ Damage to the epiphysis of long bones as a result of placing inappropriate products and incorrectly sized spinal system implants during surgery,
- ✓ Patients should stay away from strong magnetic fields.
- ✓ The patient should immediately consult his physician when he hears abnormal sounds, pain, or increased temperature in the surgery area.

**Note:** Additional surgery may be required to correct these potential drawbacks.

#### **Intended User Profile**

Ortimplant Spinal System implants should only be used by healthcare professionals.

Surgical procedures should only be performed by medical specialists or medical residents under the responsibility of specialists who have received adequate training in the anatomical structure or formation where the surgery will be performed and are familiar with surgical techniques.

Before performing a surgical procedure, surgical literature on techniques, complications, and hazards should be reviewed. You should carefully read all operating instructions regarding product safety features before use. All necessary materials and documents must be provided by the authorized distributor.

#### **General Safety Warnings**

- ! Read the User Manual before use and take into account the warnings.
- ! The products are NOT STERILE! Before first use, sterilize according to the instructions specified in this manual.
- ! Do not use it for purposes other than its intended use.
- ! Spinal system implants should not be used for purposes other than those stated above.
- ! This device is part of Spinal System Implants used to fix cervical vertebrae, thoracic vertebrae, lumbar vertebrae. It is extremely important to protect the spinal cord, main nerve roots, main arteries and other vessels during the application. Otherwise, death or serious injuries may occur.
- ! Choosing the appropriate device, appropriate accessory and appropriate application tool set for the patient is very important. Improper use or misalignment of components may cause loss, bending, breakage, or cracking of the device or bones.
- ! Spinal implants should not be placed at the fracture line. The screw threads should fit snugly into the bone and the screw length should be long enough to allow telescopic movement in resorption of the fracture surface. However, it should be adjusted so that it does not protrude too much from the bone cortex and cause vascular nerve injury or damage to the joint surfaces.
- ! Do not mix incompatible metals.
- ! The device is made of metal materials and does not contain waste materials.

#### **Precautions**

Spinal System implants should not be reused. Loading too early may cause devices to stop working. In order to prevent the implant from causing discomfort during use, check the implants and application sets (in terms of suitability and full functionality) before use. Precautions to be taken before, during and after other procedures are listed below.

#### **Operation**

##### **Patient and Implant Selection:**

Only patients with indications and no absolute contraindications should undergo surgical procedure. However, the final decision and responsibility belongs to the surgeon who will perform the surgery.

Choosing the appropriate size, shape and design for each patient is important for the procedure to be successful. Metallic surgical implants are subjected to repeated loads during use and are limited in shape, size and design. Unless care is taken in selecting the appropriate implant for the patient, unless the implant is placed properly, and if no precautions are taken to reduce the stress on the implant, such stresses can result in buckling and device loosening, metal fatigue, and eventual fracture before healing. These situations may result in device damage or premature removal.

**Before Operation:**

- ✓ Only patients who meet the criteria specified in the indications should be selected.
- ✓ The use of Spinal System implants should be avoided in patients who have or are prone to the conditions listed in the contraindications.
- ✓ Care should be taken to transport and store implant components. Spinal System implants should not be scratched or damaged. Implants should be protected during storage against aggressive environmental conditions. Spinal System implants packages must be undamaged.
- ✓ Normally, Spinal System implants of appropriate size and number should be used. Additionally, excess stock and variety should be kept ready during surgery, especially in the operating room.
- ✓ The surgeon must become familiar with the mechanical parts of the delivery sets, implants, and various other components before using the surgical equipment, and must set up and check the system to verify that all parts of the implants, delivery sets, and other components, and necessary instruments are present and working harmoniously before the procedure begins. Spinal System implants cannot be completed with components made by other manufacturers. Different types of metal should never be used together.
- ✓ Components and tools must be clean and sterile before use. Additional sterile materials (implant size, type alternatives, application sets, hospital orthopedic surgery sets, etc.) should be available to prevent undesirable situations.

**During the operation:**

- ✓ Extreme care should be taken in the surgical approach to the areas through which the spinal cord, nerve roots and peripheral nervous system pass. Damage to nerves leads to loss of neurological functions. During the surgical procedure, maximum attention should be paid to intracranial, intrathoracic, intrapelvic organs, arterial and venous areas and these structures should be protected. Arterial or venous injuries can result in death.
- ✓ Breakage, slippage, or misuse of instrument sets or implant components may result in injury to the patient or staff.
- ✓ Spinal system implants and accessories should not be bent repeatedly. Since scratches or nicks on the implant may reduce the functional integrity of the implant, due care should be taken by the surgeon to avoid use of the implant in such cases.
- ✓ Whenever possible or necessary, an imaging system should be used to facilitate surgery.
- ✓ Do not use bone bonding agents and solvents, as it will be difficult to separate the implant components from each other and their safety and effectiveness have not been proven. Drills and cutters used during drilling and cutting operations cause high heat due to friction. Heat accumulation during surgery can cause neurological injuries and local bone tissue death.
- ✓ Before suturing the soft tissue, change the implant position if there is an inappropriate implant position..

**After the operation:**

- ✓ The post-operative instructions and warnings given to the patient by the surgeon and the patient complaints associated with them are extremely important;
- ✓ The surgeon should inform the patient to avoid placing excessive regional load on the operation area. In such cases, before bone union is completed, the patient should be informed that the Spinal System implants may bend and bend, and the fracture position may be disrupted. The patient should be informed by the surgeon about the risk of reoperation under these conditions.

- ✓ The surgeon should inform the patient about contraindications that may cause the device to loosen and/or break. If the patient is extremely elderly, frail, or mobile, the risk of bending, loosening, and breaking of the temporary fixation device may be increased during the postoperative rehabilitation period. The patient should be warned to avoid falls and sudden jolts in different positions.
- ✓ To achieve maximum surgical success, neither the patient nor the device should be exposed to mechanical vibration or shock that could cause the device to loosen. The patient should be informed about this possibility and, in particular, his/her participation in any sports activities should be restricted. The patient should be advised to be accompanied by those nearby and to use auxiliary support devices (crutches, walkers, orthoses, etc.) while standing up. The patient should especially be advised to avoid turning movements. The patient should be warned not to drink or consume a product containing nicotine, alcohol, or steroids during the bone healing process.
- ✓ The patient should be warned about the loss of movement in the surgical area and informed about how to compensate for the permanent physical limitation in body movement.
- ✓ Nonunion or delayed union of the fracture; creates excessive and repetitive stress on the implant that stabilizes the fracture. Excessive load may cause the metal device to bend, loosen, or collapse due to fatigue. It is important not to move the surgical area until appropriate bone union is achieved by x-ray examination. If the device is loose, bent and/or warped, it must be replaced or removed without serious injury. The patient should be warned of these dangers and monitored closely until the bone consolidates.
- ✓ Especially in high-risk patients, prophylactic antibiotic use is precautionary before subsequent surgical procedures (such as dental surgery).
- ✓ Implant fixation devices are designed to provide stability at the fracture site during normal fracture healing. Once the bone is fused, these devices have no functional effect and can be removed. The final decision to remove the device is up to the surgeon and the patient. Many patients may experience one or more of the following complications;
  - Corrosion, local tissue reaction or pain,
  - Impairment of implant position (leading to possible injuries),
  - Extra risk of injury due to trauma to the surgical area after surgery,
  - Bending, loosening or breaking of the implant in the surgery area, which may make removal difficult,
  - Pain, discomfort or abnormal movement while the implant is in the body,
  - High risk of possible infection,
  - Bone loss caused by excessive pressure on the bone.

### Removing the Implant:

The length of time the device remains on the patient is determined by the surgeon, taking into account the progress of the union and the duration of treatment. If the implants are not placed properly, they may be exposed. Loosening of screws may cause infection. Due to such problems, implants need to be removed. Spinal System implants are removed from the body by loosening the screws and using appropriate surgical instruments.

For detailed information about removing Spinal System implants, please contact your doctor.

**Note:** Ortimplant recommends removing the implant after the fracture has healed to prevent refracture of the bone and other complications. Spinal System implants and their components should not be reused. It is not possible to use a device from one patient to another and should not even be considered.

### Surgical Technique

A separate surgical technique guide is provided to explain the procedure for Spinal System implants during the surgical procedure. The surgeon should be familiar with the surgical procedure before using the products. It is also the surgeon's

responsibility to consult with colleagues experienced in the procedures and to become familiar with relevant publications before use. For the correct application method, please review the surgical technique document provided by Ortimeplant.

### Sterilization

Steam Sterilization Method has been validated and is recommended as the sterilization method for our products.

- ! The steam sterilizer must comply with the EN 285 standard and be validated according to the EN ISO 17665 standard.
- ! It must be ensured that the maximum allowable loading is not exceeded according to the sterilizer manufacturer's instructions.
- ! The properties of the water to be used in the sterilization process must comply with EN 285 and ISO 17665 standards.

Sterilization Method	Temperature	Time
Steam Sterilization	134 °C	5 min

### Packaging and Storage Conditions

All Ortimeplant Spinal System implants are cleaned and disinfected according to standards-compliant methods. Cleaning and decontamination involves rinsing with neutral cleaning agents followed by washing with distilled water.

Afterwards, all products are packaged in the clean room.

**Note:** It should be noted that the packages containing Ortimeplant Spinal System implants are not damaged. If a delivery note or invoice is used, check that all products are complete, including application sets, before use. Damaged packages and products should not be used and should be returned to Ortimeplant or the distributor.

Ortimeplant Spinal System implants should be stored in an environment with humidity below 80% and the environment should not contain corrosive gas.

### Disposal Conditions

Disposal of the products must be carried out in accordance with the national legislation of the country where the products are used.

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**Manufacturer:** Ortimplant Ortopedi ve Tıbbi Aletler Tic. Ltd. Şti.