YENİDEN KULLANILABİLİR CERRAHİ EL ALETLERİ - KULLANIM KILAVUZU

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 Protect from sunlight!

 Manufacturer

 CE marking according to EU 2017/745 Medical Device Regulation

References

(EU) 2017/745 Medical Device Regulation

EN ISO 15223-1 Medical devices - Symbols to be used with information to be supplied by the manufacturer - Part 1: General requirements

EN ISO 17664-1 Processing of health care products - Information to be provided by the medical device manufacturer for the processing of medical devices - Part 1: Critical and semi-critical medical devices

EN ISO 17665-1 Sterilization of health care products - Moist heat - Part 1: Requirements for the development, validation and routine control of a sterilization process for medical devices

EN ISO 17665-2 Sterilization of health care products - Moist heat - Part 2: Guidance on the application of ISO 17665-1

EN ISO 15883-1 Washer-disinfectors - Part 1: General requirements, terms and definitions and tests

EN 285 Sterilization. Steam sterilizers. Large sterilizers

EN 14885 Chemical disinfectants and antiseptics - Application of European standards for chemical disinfectants and antiseptics

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www.das.org.tr (Disinfection Antisepsis Sterilization Association)

Intended Use and Indication Information

Reusable surgical instruments;

- In order to perform surgical functions such as cutting, retracting, scraping, hammering, sucking, aspirating, holding, drilling and similar functions without being connected to an active device, and
- These are instruments that are produced for reuse by following appropriate procedures such as cleaning, disinfection and sterilization.

Contraindication Information

The devices have no known contraindications. However, reusable surgical instruments are contraindicated for all applications other than those specified in the intended use and indication. Conditions that may affect the success of the operation, such as injury or disease, should be evaluated by the practitioner. Risks to anatomical structures in the planned area should be taken into account.

Intended User Profile

Instruments should be used by professionals with health or medical training.

Intended Population Information

There is no specific population for reusable surgical instruments. It can be used for all types of patients.

Instrument preferences suitable for the patient are determined by expert operators.

General Safety Warnings

- ! Read the Instructions for Use before use and take into account the warnings.
- ! Keep the Instructions for Use throughout the life of the product.

- ! The products are NON-STERILE. Before first and each use, clean and sterilize according to the instructions specified in this manual.
- ! Do not use for purposes other than intended.
- ! Check/Inspect the instruments before each use. See. **Control and Review Process**

Warnings Regarding Cleaning, Disinfection and Sterilization

- ! For cleaning, disinfection and sterilization of products, comply with national and international standards, directives and clinical hygiene instructions.
- ! Use the products used for cleaning and disinfection as recommended by their manufacturers.
- ! Perform cleaning procedures (manual or machine) before the first sterilization of newly purchased instruments.
- ! For patients with diagnosed or suspected Creutzfeldt-Jakob disease (CJD) or its variants, comply with national regulations regarding disinfection and sterilization of products.
- ! Use validated methods, calibrated autoclaves and washing machines for all operations.
- ! Use Personal Protective Equipment (PPE) when working with contaminated instruments.
- ! Instead of manual cleaning, machine cleaning should be preferred as it gives more reliable results.
- ! Do not drop, knock or apply excessive force to the instruments during cleaning and disinfection processes.

Cleaning and Care Products

- ! Avoid using metal brushes during manual cleaning as they may cause corrosion!

- ! Avoid the use of chlorine and chlorine-containing substances (saline solution, bleaches, ringer's solution, etc.) as they may cause corrosion!
- ! Avoid the use of alkylamine-containing chemicals in products containing silicone as they may cause hardening.
- ! Only use cleaning agents with approved effectiveness (VAH or FDA approval or CE marking).
- ! Use disinfectants that comply with EN 14885 standard or local regulations.
- ! You can use soft and lint-free fabrics, paper fabric, plastic, soft bristle brushes and water spray guns.
- ! Do not use cleaning products containing aldehydes (e.g. glutaraldehyde) due to their protein-fixing properties.
- ! Prepare your cleaning solutions daily.
- ! For drying, use compressed air guns instead of drying methods with cloth.

- ! Ultrasonic washing is recommended before manual and machine washing, especially for lumen/pipe and jointed/articulated instruments.
- ! Detergents suitable for ultrasonic washing should be preferred.

- ! Ultrasonic washing baths are not recommended for products containing aluminum and anodized aluminum.
- ! After ultrasonic washing, the instruments should be rinsed until there is no detergent left on them.
- ! During cleaning, disinfection and sterilization, instruments must be handled properly and in a manner that does not come into contact with each other.

Characteristics of the Water to be Used

Limit values of water to be used in the final rinse of cleaning, disinfection and sterilization processes must comply with EN 285 and ISO 17665 standards.

Use of inappropriate water in pre-cleaning, cleaning and sterilization processes;

- ! water stains,
- ! various corrosions,
- ! lime and salt stains,
- ! thin layers of various colors, and
- ! rust stains

can cause the formation.

Softened water: Partially free of minerals that add hardness to water (such as calcium and magnesium cations).

Demineralized water: Water is purified from all mineral content using various methods (reverse osmosis, distillation, etc.).

Cleaning, Disinfection and Sterilization Instructions

E. First Procedures After Use

- ! Clean blood, tissue and mucous residues from the instrument with softened water and disposable cloths.
- ! Clean the surfaces that cannot be reached with pipe/lumen instruments from dirt by spraying water with disposable syringes.
- ! Dry the instruments with compressed air guns and place them in closed transport containers in a dry and orderly manner.

F. Transportation Procedures

- ! Transporting and storing instruments wet and/or without being completely removed from cleaning solutions increases the risk of corrosion and microbiological proliferation.
- ! Transfer the instruments in a dry manner.
- ! Take care to include dry transported instruments in the cleaning, disinfection and sterilization process after 6 hours at the latest.
- ! Transport instruments in closed containers to prevent the risk of contamination in the cleaning, disinfection and sterilization unit.

G. Pre-Cleaning Process

- ! Do not apply any temperature above 25 °C during the pre-cleaning process.
- ! First, rinse the instruments with softened water at room temperature to remove coarse dirt.
- ! Include disassembled instruments in the process.
- ! It is also recommended to use ultrasonic washing to increase the cleaning effectiveness of heavily soiled instruments, lumen/tubular instruments, micro instruments and jointed instruments.

- ! Only for products made of stainless steel, you can use an immersion pool prepared with 3% hydrogen peroxide (H₂O₂) solution to clean the clotted and stuck residues on the instrument.

H. Manual Cleaning / Disinfection

- ⚠ ! Do not apply any temperature above 45 °C during the Disinfection / Manual Cleaning process. High temperature can cause protein denaturalization.
- ⚠ ! Use a solution that does not have protein stabilizing properties and does not contain alcohol, aldehydes, phenols or QUATs.
- ⚠ ! Perform all scrubbing operations below the surface of the solution to prevent air bubbles. Do not manually wash cutting instruments or pointed instruments that could harm the applicator.

Order of Process	Processes	Water Quality	Temperature	Time
1	Cleaning/Disinfection	Softened water	Room temperature	15 min
2	First Rinsing	Softened water	Room temperature	1 min
3	Disinfection	Softened water	Room temperature	15 min
4	Last Rinsing	Demineralized water	Room temperature	1 min
5	Rinsing	-	Room temperature	-

1. Process: Cleaning/Disinfection

- ✓ 0.5% cleaning/disinfection dilution should be created. (Put 25mL solution into 5L water and mix.)
- ✓ All instruments should be immersed in the solution and shaken from side to side to prevent air bubbles from forming.
- ✓ Move the articulated instruments to ensure that the solution comes into contact with all surfaces.
- ✓ Spray the solution onto the inner surfaces and holes of the lumen/tube instruments using a disposable syringe. You can also brush the inside of the instruments by using cylindrical brushes suitable for the inner diameters of these instruments.
- ✓ While the instruments are in the solution, brush them with a soft-bristled cleaning brush until the dirt on their surfaces is removed.
- ✓ Take care to expose the solution to areas of the instruments that are difficult to contact with the solution.
- ✓ For instruments of a size that cannot be immersed in the solution pool, you can use spray mechanisms to clean the instruments by spraying the solution on them.

2. Process: First Rinsing

- ✓ After removing the instruments from the solution, rinse them with softened water for at least 1 minute.
- ✓ Move articulated and mobile instruments during rinsing to ensure that water reaches every part of the tool.
- ✓ Rinse the inner surfaces of the lumen/tube instruments using a disposable syringe.
- ✓ Before proceeding with the disinfection phase, make sure that the excess water on the instruments is completely drained.

3. Process: Disinfection

- ✓ Create a 2.5% dilution. (Put 125mL of water into 5L of water and mix.)
- ✓ Immerse the instruments completely in the disinfection solution.
- ✓ Move the parts of the articulated instruments at least 5 times to ensure that the solution comes into contact with all surfaces.
- ✓ Spray the solution onto the inner surfaces and holes of the lumen/tube instruments using a disposable syringe. You can also brush the inside of the instruments by using cylindrical brushes suitable for the inner diameters of these instruments.

4. Process: Last Rinsing

- ✓ Rinse the instruments with demineralized water for at least 10 seconds.
- ✓ Move the parts of the articulated instruments at least 5 times to ensure that the water comes into contact with all surfaces.
- ✓ Spray water on the inner surfaces and holes of the lumen/tube instruments using a disposable syringe.
- ✓ To prevent the formation of water stains, pay attention to the use of demineralized water.

5. Process: Rinsing

- ✓ Dry the instruments with the help of a compressed air gun.
- ✓ You can also use a clean, lint-free absorbent cloth.
- ✓ Take care to dry the inner surfaces and holes of the lumen/tube instruments.
- ✓ After the cleaning and disinfection processes, check the cleanliness of the product by visual inspection.
- ✓ Joint, lumen, lock, tooth, etc. Make sure that difficult-to-clean areas such as.
- ✓ If you are not sure that the instruments are completely clean, be sure to repeat the washing process.
- ✓ Maintain the cleaned instruments. See. **Maintenance Process**
- ✓ Check the instruments after maintenance operations. See. **Inspection and Check Process**

i. Automatic Cleaning / Thermal Disinfection

! Clean the instruments with cleaning materials with an alkaline value of 8-11 pH.

! Follow the manufacturer's instructions for the disinfectant you have. Use a disinfectant validated according to EN ISO 15883 standard or FDA approved.

- ✓ Place the instruments in the disinfectant.
- ✓ For effective cleaning, follow the manufacturer's instructions when placing the instruments.
- ✓ Place all instruments disassembled.
- ✓ Use trays or baskets designed for washer-disinfectant.
- ✓ Place heavy and large instruments lower.
- ✓ If the disinfectant you have has special shelves, use them according to the manufacturer's instructions.
- ✓ Place the instruments in special holders or on appropriate shelves to prevent them from being damaged during the process.
- ✓ Place the instruments in the disinfectant so that they do not come into contact with each other to avoid mechanical damage.
- ✓ Avoid overloading the disinfectant so that cleaning solutions come into contact with all surfaces of the

instruments.

- ✓ Remove the instruments from the disinfectant immediately after the disinfection program is completed.
- ✓ The following parameters have been validated;

Order of Process	Processes	Water Quality	Temperature	Time
1	Pre-wash	Softened water	< 25 °C	3 min
2	Cleaning	Demineralized water	40 – 55 °C	10 min
3	First Rinse / Neutralization	Demineralized water	> 10 °C	2 min
4	Intermediate Rinsing	Demineralized water	> 10 °C	1 min
5	Thermal Disinfection / Final Rinsing	Demineralized water	90 °C	5 min
6	Drying	-	-	-

Maintenance Process

- ⚠ ! USP/NF, Ph. Use care products that contain ingredients that comply with pharmacopoeias such as EUR and local and international cosmetic regulations.
- ⚠ ! Do not use products containing silicone oil!
- ! Make sure the instruments are completely cleaned before maintenance..
- ! Care products; Make sure that it is paraffin-based, biocompatible and suitable for sterilization (vapor permeable).
- ! Be careful to apply the care product directly to the jointed and threaded areas of the instruments.
- ! Operate the moving mechanisms several times to spread the care product evenly.
- ! Remove excess care product with a lint-free cloth.

Inspection and Check Process

- ! Branding on the product must be legible.
- ! There should be no cracks, breaks, bending, abrasions, burrs or discoloration, stains, rust or corrosion on the surface of the instruments.
- ! There should be no loose, bent, broken, cracked, worn or broken parts.
- ! There should be no color loss or deterioration in coating integrity on coated, painted and insulated coated products.
- ! Instruments with lumen/pipe should be checked for blockages with a suitable cleaning brush. If a blockage is detected, reprocessing must be repeated.
- ! Cutting and drilling instruments must not be blunt.
- ! Moving parts must be able to move freely without binding or rubbing.
- ! Products with spring parts must regain their initial form after use.
- ! In assembled products, matching parts must be compatible with each other.

NOTE: If any of the above checks are not verified, never use the product. In such negative situations, please contact us. Damaged instruments should only be serviced by us.

Sterilization Process

 Steam Sterilization Method has been validated and is recommended as the sterilization method for our products.

- ! The steam sterilizer must comply with the EN 285 standard and be validated according to the EN ISO 17665 standard.
- ! It must be ensured that the maximum permissible loading is not exceeded according to the sterilizer manufacturer's instructions.
- ! The properties of the water to be used in the sterilization process must comply with EN 285 and ISO 17665 standards.
- ! Ethylene oxide or other sterilization methods have not been validated by our company.

Sterilization Method	Temperature	Time
Steam Sterilization	134 °C	5 min

Storage Conditions

-  ! Newly purchased surgical instruments should not be stored in dusty and humid environments.
- ! Instruments can release corrosive vapors when in direct contact with chemicals. For this reason, surgical instruments should not be stored together with chemicals and in the same environment.
-  ! Sterile packaged surgical instruments should be stored in designated areas away from direct sunlight, well ventilated, not excessively hot/humid, and where necessary precautions are taken against insects and rodents.

Disposal Conditions

Disposal of the products must be carried out in accordance with the national legislation of the country where the products are used.

Technical service

-  ! The warranty period of the instruments is 2 years.
-  ! We kindly request that you send the products to be sent to us for service in a packaged and sterile manner!

In case of any adverse event related to the device, the manufacturer or local authorities should be reported!

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Manufacturer: Ortimplant Ortopedi ve Tibbi Aletler Tic. Ltd. Şti.